Quick Start Guide



What's in the box?





Sensor Charge Cable and Cable Clip




Medium Boot (Size 4 Strap Pre-Attached)



5W Sensor Charge Adapter




Small Boot (Size 3 Strap Pre-Attached)

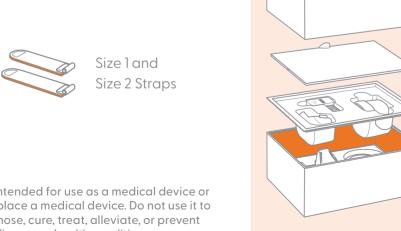



Not intended for use as a medical device or to replace a medical device. Do not use it to diagnose, cure, treat, alleviate, or prevent any disease or health condition.

Charge the sensor

Plug the Sensor Magnetic Charge Cable into the 5W power adapter. Connect the magnetic end of the cable to the sensor. Fully charge the sensor before use.

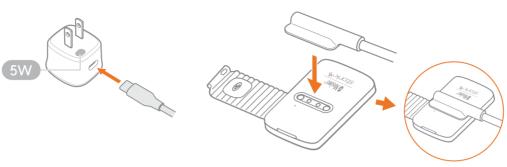

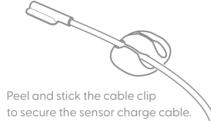

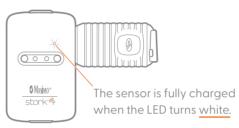

Pair the sensor to the app

Download the Stork app and follow instructions.








The app screens shown in this document might differ from the actual ones displayed in the app.


Add straps to sensor and boot




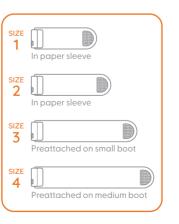


Remove sizes 1 and 2

from paper sleeve.

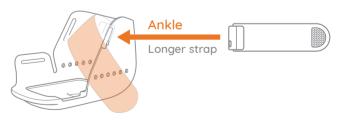


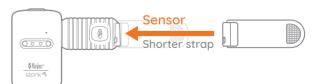

Insert Sensor Into Small Boot

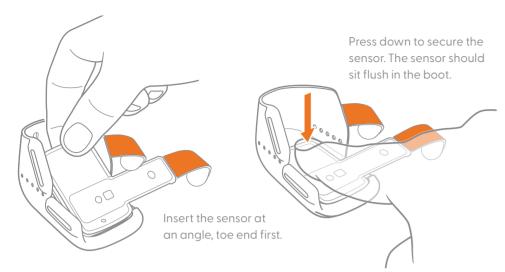


Apply Boot to Either Foot

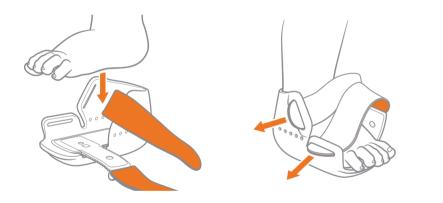

Check the Strap Length




Boot Sizing (Using Small Boot As Example) _

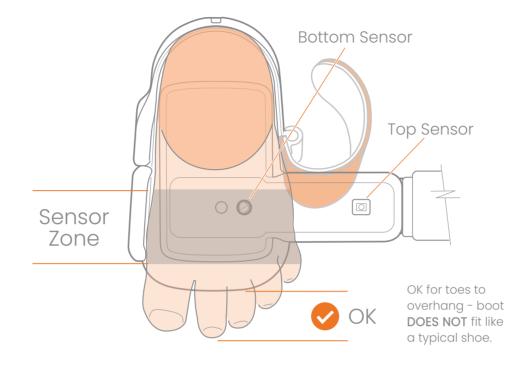

IMPORTANT NOTE


Make sure all sensors are making good contact with the baby's skin to ensure correct measurement.

Strap Tightness _

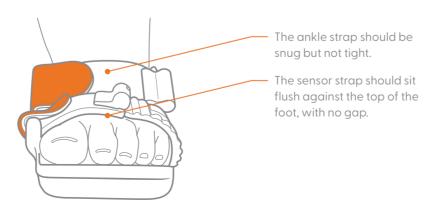

Boot and sensor maintenance

Remove Sensor

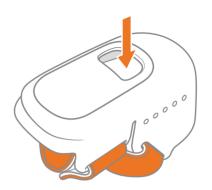

To remove the sensor, push it from the opening at the bottom of the boot.

Cleaning Sensor and Boot _

Remove the sensor and place just the boot (and the straps if desired) in the washing machine.



<u>Do not</u> put the sensor in the washing machine

Dampen a cloth with water to wipe the components of the Stork System (sensor is shown in the example). Bleach wipes or 70% isopropyl alcohol can also be used.

Do not use household cleaning agents.



OWNER'S MANUAL

Refer to the online manual for other functions, information, and operation details.

masimostork.com/vitals-plus-user-manual

